

USER MANUAL

TKEZYM Manual Patient Trolley



DEALER: This manual **MUST** be given to the user of the product.

All rights reserved. Reproduction and copying in whole or in part prohibited. We reserve the right to carry out changes by reason of technical developments. The contents of this document are subject to change without prior notice. Customers are advised to consult their sales representative before placing an order.



TO DEAR CUSTOMERS

Thank you for deciding to purchase our company's product and we believe you will be satisfied with it throughout the time of its use. Due to the very careful selection of materials, the latest production technology and careful work of our staff, your bed achieves the parameters that guarantee high quality, reliability and constant utility value.



WARNING

DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment- otherwise, injury or damage may occur.

The initial set up of this bed must be performed by a qualified technician. Procedures other than those described in this manual must be performed by a qualified technician.



Copyright

This user's manual may only be made available to third parties with the express written agreement of Jiangsu Yongxin Medical-use Facilities Making Co., Ltd.. All documentation is subject to copyright. Unless expressly permitted, the transfer and duplication of documentation, either whole or in part, as well as its exploitation and/or publication, are forbidden. Infringements of copyright are punishable in law and oblige transgressors to pay damages. The company reserves the right to exercise all its prerogatives in commercial law.

1. PURPOSE

This trolley is used for transfer patients.

2. STRUCTURAL COMPOSITION



4. ESSENTIAL PARAMETER

No	Model name	Total length	Total width	Highest position of the trolley surface to the ground	Height adjustment
1	YXZ-E-1	1970cm	640mm	86mm	≥300mm

When it is in horizontal position

No	Model name	Back section adjustment range
1	YXZ-E-1	≥70°

When trolley is in the highest and horizontal position

Loading capacity of the trolley is 250KGS

5. INSTRUCTION OF OPERATION

1. Center braking system instruction



Step the red part, the trolley will be locked

Step the green part, the trolley will be released

2. Guide wheels instruction



Step the red part, the guide wheel will be hided

Step the green part, the guide wheel will fall down to touch the ground as a guide wheel

3. Instruction of height adjustment



Shake the crank to adjust the height

4. Instruction of back section adjustment



Press the yellow part and pull it up, the back side will be up

Press the yellow part and press it down, the back side will be down

5. Instruction of the side rail



1. ABS side rail
2. Handle
3. Locking status of side rail
4. Brake of the side rail
5. Bed platform

6. SAFETY INSTRUCTION

1. Before using the trolley, you must read the manual and handle the manual in accordance with it
2. The trolley may not be used if faults have been detected on it that may injure the patient, staff or a third person, the trolley or the surrounding interior
3. The trolley may only be handled by persons who, by their knowledge or practical experience, provide a guarantee that the bed shall be handled without any problems
4. The user (operator) is obliged, before using the trolley, to become convinced of the functionality and faultlessness of the trolley
5. The trolley may be used only on even and solid floors and under the conditions set forth in the paragraph “Conditions of use”
6. The trolley may not be overloaded even for a brief period, but if this is truly necessary in specific situations (such as during reanimation), the individual parts of the mattress platform must be in the lowest position
7. When handling the mobile parts of the trolley, care must be taken that the patient, other people or objects do not get trapped during its movement
8. If a lifting pole or infusion stand is fixed to the bed during movement, lifting or tipping, increased attention must be taken to the space around the lifting pole and infusion stand so the equipment does not get damaged
9. The trolley may not be used where there is a danger of explosion or in the presence of inflammable anesthetics.

10. When repairing the bed, only original materials and components may be used; when replacing damaged parts with inappropriate materials and components, the manufacturer does not guarantee against any damage that can occur

11. An inspection must be made at least once a month for wear and tear of all mobile parts

12. The doctor treating the patient or another responsible person providing treatment decides about the use of the handrails for supporting the patient when leaving the trolley



Get in touch or call us now!

 **1300 788 928**

 info@diagmed.com.au

 www.diagmed.com.au